


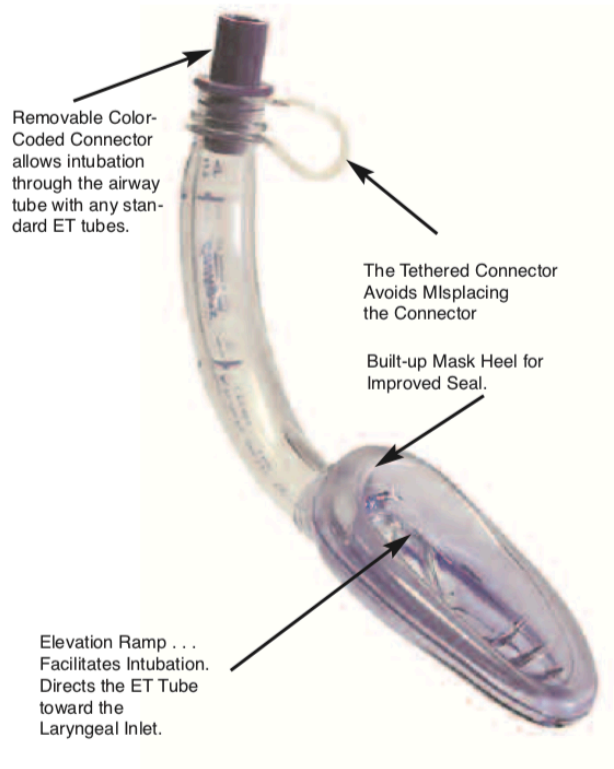

## QUICK GUIDE

### Insertion “Tips & Techniques”

- Extend the head back.
- Lubricate the air-Qsp mask cuff, back of the cuff and the ridges.
- Use a tongue depressor to lift the tongue.
- Place the air-Qsp in at a forward angle aiming for the back of the tongue and soft palette.
- Once around the corner, take out the tongue depressor, lift the mandible and push with the index finger on the top of the colored connector.
- Support the airway tube with your other fingers.
- Check the placement with Positive Pressure Ventilation (PPV), listen for breath sounds and confirm with capnography.



air-Q Sp 4.5

Distributed Exclusively By:



272 East Deerpath 302 Lake Forest, IL 60045

Registered trademark, air-Q®, property of Cookgas, LLC  
Patents U.S. 5,937,860  
US 6,422,259 B1  
US 6,960,751 B2  
US 7,331,347 B2  
US 6,762,561 B2  
US 7,357,845 B2  
US 7,790,950 B2  
Patents Canada 2,231,201  
2,371,435  
Patents UK GB2351437B  
GB407293B  
GB405599B  
GB234040B  
Other patents pending.

### Customer Support (USA)

Toll Free: 800-421-0024

Email: [customercare@salterlabs.com](mailto:customercare@salterlabs.com)

### International Support

Toll Free: 303-799-1990

Email: [international@salterlabs.com](mailto:international@salterlabs.com)